Infusomat[®] Space Line SafeSet

Infusomat[®] Space Line

GB	Instructions for use
DE	Gebrauchsanweisung
BG	Инструкция за употреба
CN	使用说明
CZ	Návod k použití
DK	Brugsanvisning
EE	Kasutusjuhised
ES	Instrucciones de uso
FI	Käyttöohje
FR	Mode d'emploi
GR	Οδηγίες χρήσης
HR	Upute za uporabu
HU	Használati utasítás
ID	Petunjuk penggunaan
IT	Istruzioni per l'uso
KR	사용 지침
KZ	Қолдану жөніндегі нұсқаулық
LT	Naudojimo instrukcija
LV	Lietošanas instrukcija
NL	Gebruikersinformatie
NO	Bruksanvisning
PL	Instrukcja użytkowania
PT	Instruções de utilização
RO	Instrucțiuni de utilizare
RS	Uputstvo za upotrebu
RU	Инструкция по применению
SE	Bruksanvisning
SI	Navodila za uporabo
SK	Návod na použitie
TH	คำแนะนำในการใช้
TR	Kullanım Kılavuzu
UA	Інструкції для застосування
VN	Hướng dẫn sử dụng

B | BRAUN

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Infusomat® Space Line

GB Type IV-Standard

Indications: For standard Infusion and for the infusion of parenteral fluids (e. g. Parenteral Infusion Solutions and Lipids)

Contraindications: Not for the use with: enteral nutrition solutions, light sensitive drugs, blood and blood products.

Type Light Protect

Indications: For light sensitive drugs.

Contraindications: Not for the use with: enteral nutrition solutions, blood and blood products.

Type Oncology

Indications: For standard Infusion solutions and for low soluble IV-Solutions and for cytostatic solutions.

Contraindications: Not for the use with blood and blood products, lipid emulsions, light sensitive drugs, enteral nutrition solutions.

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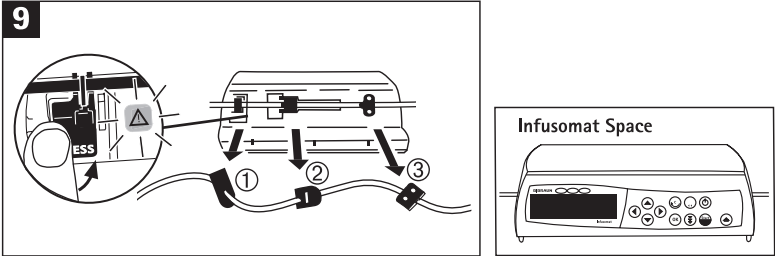
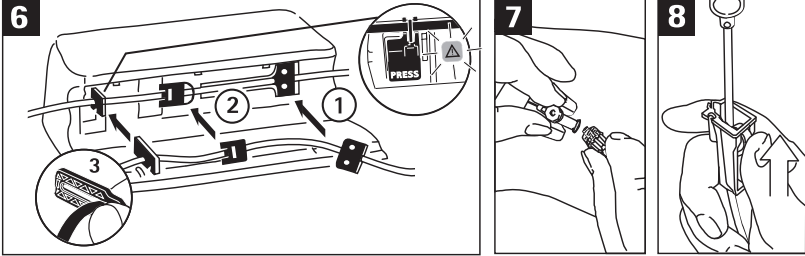
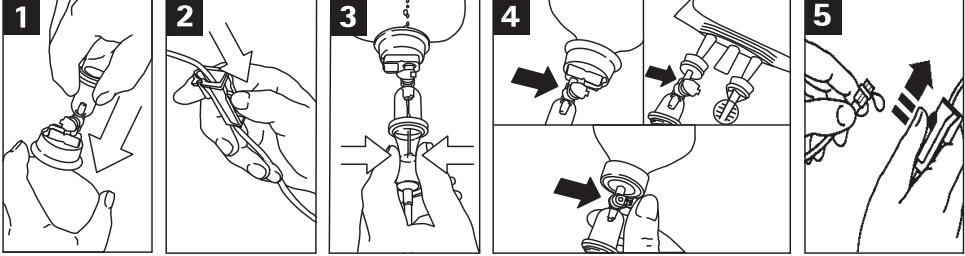
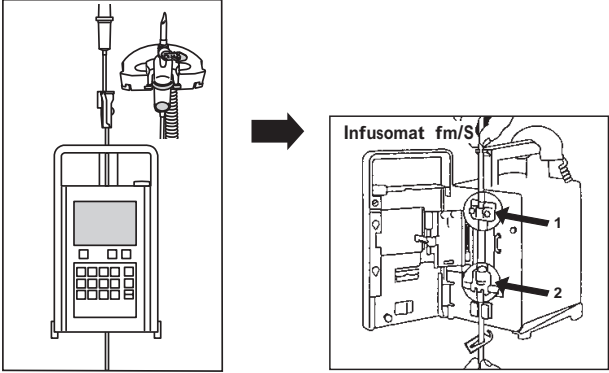
Warning notices

GB Warning

Re-use of single-use devices creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.

Sterile, non-pyrogenic.

Do not use if protective caps are loose or missing or if packaging is damaged.

Caution! If a pneumatic cuff is used during infusion, remove air from the container beforehand. Observe general safety precautions!

Please observe Instructions for Use of B.Braun Infusion pumps and mind instructions for parallel infusions.

When using Infusomat Space and Infusomat fmS without a drip sensor, use a volume setting smaller / equal the filling volume of the container.

If Infusomat Space Line with NEUTRAPUR tubing is used please select respective „Space Line NEUTRAPUR“ setting.

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B. Braun Melsungen AG